

## Supplies

1. Bowl
2. Clean tube brush
3. Gauze dressing

Inner cannulas should be cleaned at least daily and more often as needed to remove mucus build up and keep tube surfaces clean.



1 Prepare a bowl with tap water (drinkable), or if needed, sterile or distilled water.




2 Rinse your used inner cannula in the bowl. This can also be done under tap water.



3 Bend brush to align with curvature of cannula if using a rigid tube.



4 Clean the inner cannula by inserting the brush into the distal end of the tube. Brush to remove mucous.

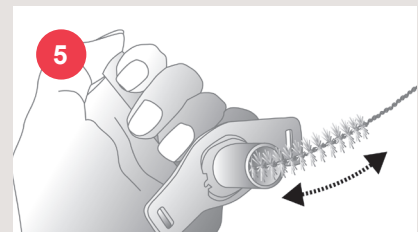

5 If brushing from the connector end, close the distal opening with your thumb to prevent the brush from coming out and damaging the cannula.

Also brush outside of inner cannula.



6 Rinse inner cannula in a clean bowl of water or under lukewarm tap water.



7 Leave cannula to air dry. Place in clean and dry storage container until next use.



8 Rinse the tube brush in clean water after use.

## Attention

Never insert the brush into the tracheostoma itself to clean. The inner cannula must always be removed from the outer tube prior to cleaning.



Watch video instructions here

**Head office:**

Atos Medical AB  
Hyllie Boulevard 17  
SE-215 32 Malmö, Sweden  
Tel: +46 (0) 415 198 00  
Email: [info@atosmedical.com](mailto:info@atosmedical.com)

**Manufacturer:**

Atos Medical AB  
Kraftgatan 8  
SE-242 35 Hörby, Sweden

**Manufacturer:**

Tracoe Medical GmbH  
Reichelsheimer Straße 1/3  
55268 Nieder-Olm, Germany

© Coloplast A/S. All rights reserved. Atos and the Atos Medical logo are trademarks of Coloplast A/S.

The information contained in this material does not replace the Instructions for Use of the products demonstrated. You must see your physician, clinician, or qualified caregiver for advice on your condition and on products that may be appropriate for you.

MC3021-TcEN 202305