




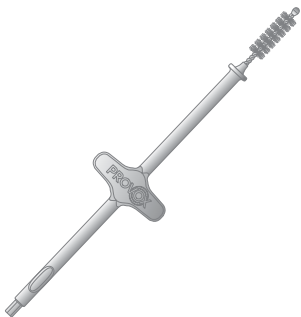
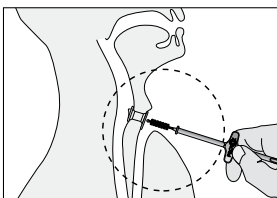
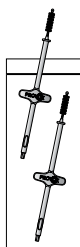
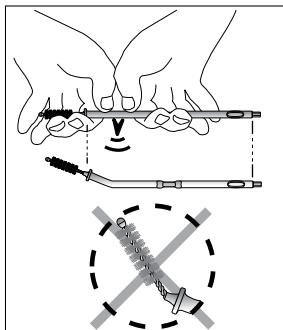
# PROVOX<sup>®</sup> Brush

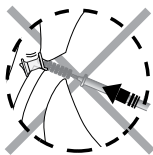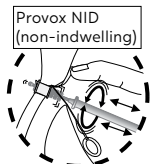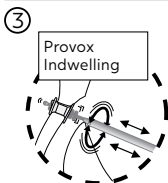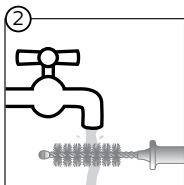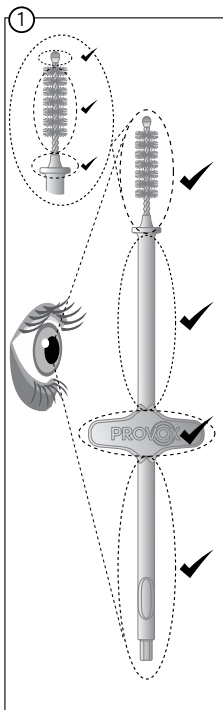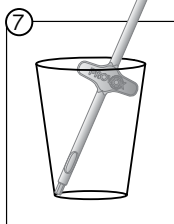
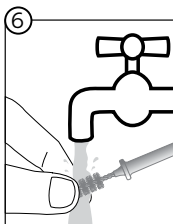
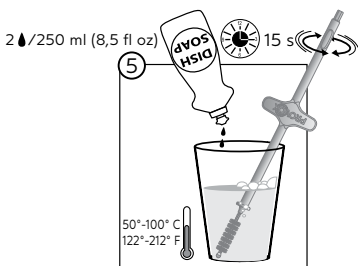
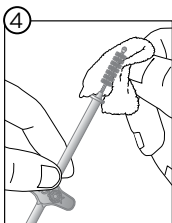
1	2	3	4	5	6	7
8	9	10	11	12	13	14
15	16	17	18	19	20	21
22	23	24	25	26	27	28
29	30					

8

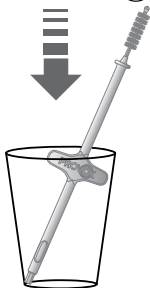
70 % Ethanol  10 min

or

70 % Isopropylalcohol  10 min

or

3 % Hydrogenperoxide  60 min

## **Disclaimer**

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

## **Patents and trademarks**

Provox® is a registered trademark owned by Atos Medical AB, Sweden. For information about protective rights (e.g. patents), please refer to our web page [www.atosmedical.com/patents](http://www.atosmedical.com/patents)

## Contents

EN - ENGLISH.....	8
LV - LATVIEŠU .....	9

## Intended Use

Provox Brush is a single patient use device intended for cleaning Provox Voice Prosthesis, for insertion of Provox Plug and Provox Vega Plug, for applying lubricant and anti-candida agents and for cleaning of fenestration holes on Provox LaryTubes. The product is intended for use by the patient.

## Before use

Always inspect the Provox Brush (fig. 1) to make sure that it is not broken or worn out.

Before each use, make sure that

- the protective tip is not cracked or loose.
- the bristles do not appear worn or are loose.
- the wires are not bent or broken.
- the entire handle is not cracked or broken and that the safety wings have not been removed.

If any defects are found, the brush must be discarded.

## Cleaning and disinfection

Clean the Provox Brush after each use according to fig. 4-7 using drinking water.

Disinfect the Provox Brush (fig. 8) with one of the following methods:

- Ethanol 70% for 10 minutes
- Isopropylalcohol 70% for 10 minutes
- Hydrogenperoxide 3% for 60 minutes

Do not use the device until it is completely dry

**Note:** Always clean and disinfect before use.

## Precaution

- The device is intended for single patient use only. Reuse in another patient may cause cross contamination.
- Do not clean or disinfect by any other method as this might cause product damage and injury.

## Disposal

Always follow medical practice and national requirements regarding biohazard when disposing of a used medical device.

## Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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© Atos Medical AB, Sweden.