
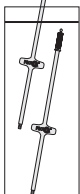
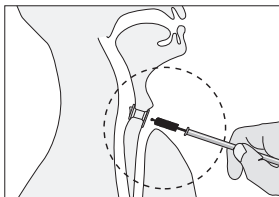
# PROVOX Brush

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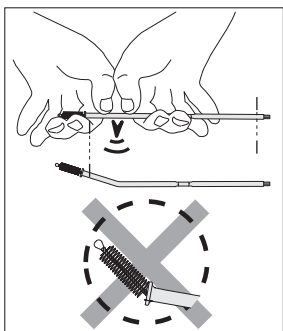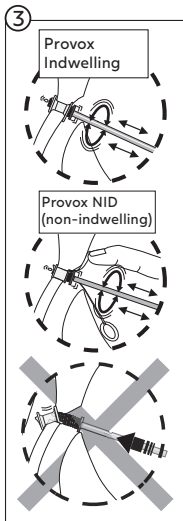
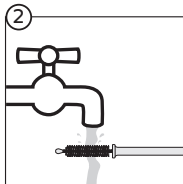
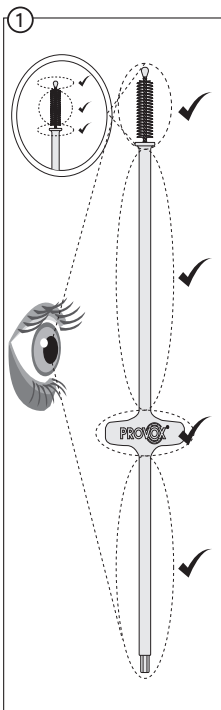
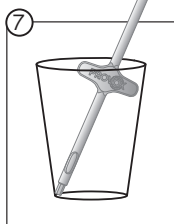
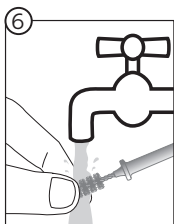
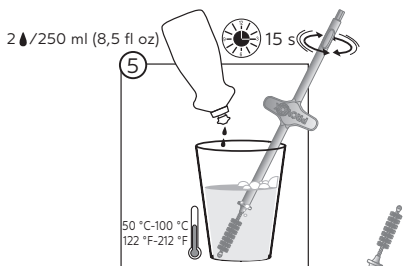
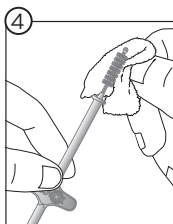
1	2	3	4	5	6	7
8	9	10	11	12	13	14
15	16	17	18	19	20	21
22	23	24	25	26	27	28
29	30					

8

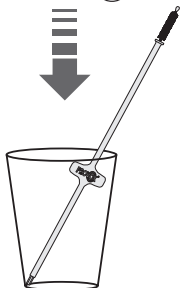
70% Ethanol  10 min

or

70% Isopropylalcohol  10 min

or

3% Hydrogenperoxide  60 min

## **Disclaimer**

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

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# ENGLISH

## **Intended Use**

Provox Brush is a single patient use device intended for cleaning Provox Voice Prosthesis, for insertion of Provox Plug and Provox Vega Plug, for applying lubricant and anti-candida agents, and for cleaning of fenestration holes on Provox LaryTubes. The product is intended for use by the patient.

## **Before Use**

Always inspect the Provox Brush (fig. 1) to make sure that it is not broken or worn out.

Before each use, make sure that

- the protective tip is not cracked or loose.
- the bristles do not appear worn or are loose.
- the wires are not bent or broken.
- the entire handle is not cracked or broken and that the safety wings have not been removed.

If any defects are found, the brush must be discarded.

## **Cleaning and Disinfection**

Clean the Provox Brush after each use according to fig. 4-7 using drinking water.

Disinfect the Provox Brush (fig. 8) with one of the following methods:

- Ethanol 70% for 10 minutes
  - Isopropylalcohol 70% for 10 minutes
  - Hydrogenperoxide 3% for 60 minutes
- Do not use the device until it is completely dry.

**Note:** Always clean and disinfect before use.

## **Precautions:**

- The device is intended for single patient use only. Reuse in another patient may cause cross contamination.
- Do not clean or disinfect by any other method as this might cause product damage and injury.

## Disposal

Always follow medical practice and national requirements regarding biohazard when disposing of a used medical device.

## Product Model

Model	Specification
8404-18	Provox Brush Long
8425-18	Provox Brush Long XL

## Storage Conditions

Store the product dry and away from sunlight at room temperature. Excursions permitted between 2 °C - 42 °C.

## Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

## Legal Agent and After-Sales Service Information

Coloplast (China) Medical Devices Ltd.

Address: Unit1301-1306, 13th Floor, Building 1, No.5 Lido Huayuan Road, Chaoyang District, Beijing

Agent contact information:

Tel: 010-5920 1888

Fax: 010-5920 1898

Coloplast Customer Service Hotline

Hotline: 400 700 7668

Website: [www.coloplast.com.cn](http://www.coloplast.com.cn)

## Production Date and Validity

Please see the label for the production date; the shelf life of product is 5 years.

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联系方式：+46 (0)415 198 00 • [info@atosmedical.com](mailto:info@atosmedical.com)

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Manufacturer; 制造商

Country of manufacture with date of manufacture;  
制造国家/地区及制造日期

Use-by date; 使用期限

Batch code; 批次代码

Product reference number; 产品编号

Non-sterile; 未灭菌

Keep away from sunlight and keep dry; 怕雨, 怕晒

Storage temperature limit; 储存温度限制

Store at room temperature. Temporary deviations within the temperature range (max-min) are allowed; 室温下存放。允许温度范围内(最高-最低)的暂时偏差。

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Indicates that the product is in compliance with European legislation for medical devices; 表示本产品符合欧洲医疗器械法规

Recycling guidelines; 回收指南

11699, 03  
2024-02-14

# Atos

atosmedical.com

Atos Medical AB  
Kraftgatan 8, SE-242 35 Hörby, Sweden  
Tel: +46 (0)415 198 00 • [info@atosmedical.com](mailto:info@atosmedical.com)

Atos and the Atos logo are trademarks of Coloplast A/S.  
© 2024 Coloplast A/S. All rights reserved.