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 Packaging Unit

Purpose:

The **PRIMA-STOM SMOOTH** sealing ring secures the tracheal cannula in an uneven tracheostomy.

Description:

The **PRIMA-STOM SMOOTH** sealing ring is normally used together with a tracheal cannula. The skin side of the ring is fitted to the tracheostomy (figure 1) before the tracheal cannula is inserted. A tracheal compressive band can be worn additionally between the tube tab and the **PRIMA-STOM SMOOTH** tube sealing ring. Following its insertion, the tracheal cannula is secured with a cannula fixation strap (figure 2). The **PRIMA-STOM SMOOTH** sealing ring is a disposable item for use by one patient only and must be changed daily for reasons of hygiene. An earlier changing is indicated if the ring becomes contaminated with secretions.

Cleaning:

For reasons of hygiene the **PRIMA-STOM SMOOTH** sealing ring is not to be re-processed.

Safety Information:

Risk of infection follows if the **PRIMA-STOM SMOOTH** sealing ring is re-used or its use exceeds the prescribed duration. No allergic reactions have so far been reported.

Product disposal:

Disposal of products must be carried out in compliance with current national regulations applying to waste products.

Caution: Warranty, guarantee and liability claims against the manufacturer will be nullified if any changes are made to the product by users or if the product is used outside of the scope of applications described in these instructions.

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