# PRIMA-SECURE II

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UK Instructions for use

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 Packaging Unit

**Purpose:**

**PRIMA-SECURE II** secures the inner cannula of the Care Cannula Series against this slipping out of the outer cannula if, for example, a respiratory tube has been connected to the inner cannula.

**Description:**

See illustrations 1 to 3

**Cleaning:**

The **PRIMA-SECURE II** is a re-usable item. Cleaning is to be carried out without chemical additives in saline solution (0.9%) at a maximum of 65 °C. We recommend subsequent disinfection with a suitable disinfectant following the instructions for use.

**Safety Information:**

The **PRIMA-SECURE II** is not to be used with cannulae from other manufacturers, as in a serious instance quick and safe disconnection cannot be guaranteed. If damage is evident or functionality impaired it must be disposed of.

**Product Disposal**

Disposal of products in germ-free condition may be carried out only in accordance with current national regulations for waste materials.

**Caution:** Warranty, guarantee and liability claims against the manufacture are nullified if changes are made to the product by users, if the product is used outside of the scope of applications described in these instructions for use or if the cleaning instructions are not complied with.

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Atos Medical AB